

Well-Root ST

Root canal sealing material



Instruction for use

CE
1639

■ **For dental use only**

Eng

Description

Well-Root ST is a convenient premixed ready-to use, injectable Calcium silicate compounds paste developed for permanent obturation of the root canal. Well-Root ST is based on a Calcium silicate compounds composition, which requires the presence of water to set and harden. Well-Root ST does not shrink during setting and demonstrates excellent physical properties.

Setting time

Setting time is 25min, measured according to ISO6876:2012 (100% humidity condition). However, in normal root canals, the setting time can be more than 2.5 hours.

Composition


- Calcium silicate compounds
- Zirconium oxide
- Filler and thickening agent

Indications

Permanent obturation of the root canal.

Recommended Procedure

1. Prepare and irrigate the root canal using standard endodontic techniques.
2. For your convenience, fix a free angle holder to the syringe as below picture.

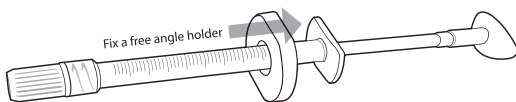

3. Remove the syringe cap from the Well-Root ST syringe. Securely attach a disposable tip with a clockwise twist to the hub of the syringe. The disposable tip is flexible and can be bent to facilitate access to the root canal.
4. Insert the tip of the syringe into the root canal no deeper than the coronal one third(1/3). Gently and smoothly dispense Well-Root ST into the apex of the root canal. A radiograph can be taken at this stage to ensure an adequate apical seal. If the placement is not sufficient, thoroughly irrigate Well-Root ST out of the root canal and repeat the procedure.
5. Continue filling the root canal with Well-Root ST while withdrawing the disposable tip. Fill the root canal completely; avoid the formation of air bubbles and overfilling of the root canal.
6. Remove excess Well-Root ST from the top of the root canal with a moist cotton pellet.
7. Place the Gutta-Percha Points into the root canal using standard endodontic techniques.



VERICOM CO., LTD.

Dental Materials

48, Toegyegongdan 1-gil, Chuncheon-Si, Gangwon-Do, Korea
vericom@vericom.co.kr overseas@vericomdental.com
www.vericom.co.kr

Copyright ©VERICOM. All right reserved.

Storage

1. Please keep the syringe cap tightly when not using Well-Root ST.
2. Avoid exposure to direct sunlight.
3. Store in a cool place.

Precautions

1. For dental use only.
2. It should be used by a dentist and dental professional. Observe the procedure and the precautions.
3. Keep it away from a reach of non-dental users including children, senior citizens and others.
4. Do not press paste with excess force, because the patient may have some acute pain.
5. Do not overfill beyond the apex.
6. In accidental contact with eyes occurs, flush immediately with a large amount of water.
7. Do not reuse the disposable tip after use.
8. Do not use after expiry date.
9. Do not use as an internal medicine.

Symbols



Use-by date



Batch code



Consult
Instructions for use



Catalogue number



Keep dry



Temperature limit



Manufacturer



Authorized representative
in the European Community



Date of manufacture



Medical device

Warranty

VERICOM CO., LTD will replace product that is proven to be defective. VERICOM CO., LTD. does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.



VERICOM CO.,LTD.

48, Toegyegongdan 1-gil, Chuncheon-Si, Gangwon-Do, Korea
Tel : +82-31-441-2881 Fax : +82-31-441-2883
E-mail vericom@vericom.co.kr overseas@vericomdental.com

www.vericom.co.kr



KTR Europe GmbH

Mergenthalerallee 77, 65760 Eschborn, Germany

VERICOM USA, INC.

22541 Aspan St, Suite A Lake Forest, CA 92630 USA
Tel : +1-949-317-3011 Fax : +1-949-317-3012