



Well-Paste

Calcium Hydroxide with Barium Sulfate

Description

Well-Paste is a temporary root canal filling material for pulp capping and infected root canal. It is also applied a good intracanal medicament. It contains Calcium Hydroxide and Barium Sulfate mainly, having good radiopacity and antibacterial characteristics (pH ≥ 12). Well-Paste is very stable without any solidification or separation and is a flowable paste premixed in a convenient syringe. Also Well-Paste has excellent accessibility to the apex and biocompatibility to the periapical tissue.

Composition

- Calcium Hydroxide
- Barium Sulfate
- Thickening Agent
- Glycol Derivatives
- Water

Intended use

To fill the root canal temporarily during endodontic treatment.

Contra-indications

Do not apply as a permanent filling material or luting material.

Indications


1. Treatment for an infected root canal
2. Pulp capping of exposed pulp
3. Disinfection after pulp extirpation treatment
4. Pulp capping of vital pulpotomy

Side effects

When Well-Paste is overfilled to beyond the apex, it may cause inflammation, tissue necrosis and nerve injury.

Recommended Procedure

1. After root canal preparation, rinse and dry the canal gently.
2. For your convenience, fix a free angle holder to the syringe as below picture.

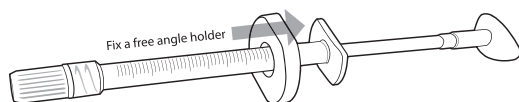

3. Insert the tip of the syringe into the root canal.
4. Apply paste by compressing the plunger gently and smoothly.
5. Withdraw the syringe gently when the paste fills the canal completely.
6. Remove excess paste on the surface.
7. Well-Paste needs to be completely removed before final obturation. Removal of Well-Paste is accomplished using appropriate irrigation solution and instrument.
8. Remove the disposable tip of syringe, close the cap tightly and store in a cool place.


Storage

1. Avoid exposure to direct sunlight.
2. Store in a cool place.

Precautions

1. For dental use only.
2. It should be used by a dentist and dental professional. Observe the procedure and the precautions.
3. Keep it away from a reach of non-dental users including children, senior citizens and others.
4. Do not press paste with excess force, the patient may have some acute pain.
5. Do not overfill beyond the apex.
6. Avoid contact with skin, eye, and mucous membrane. Contact of calcium hydroxide with the skin, eyes and mucous membrane may cause local irritation and burns. If inadvertent contact occurs, rinse with copious amounts of water and seek medical advice.
7. Please keep the syringe cap tightly when not using Well-Paste.
8. Do not reuse the disposable tip after use.
9. Do not use as an internal medicine.
10. Do not use after expiry date.
11. Do not use with the patients who have shown hypersensitivity or allergy to treatment materials.
12. Do not use if the paste has hardened, phase separation has occurred, or there are foreign substances.

Symbols

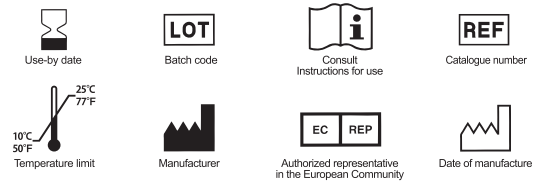

Warranty

VERICOM CO.,LTD will replace product that is proven to be defective. VERICOM CO.,LTD does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.



VERICOM CO., LTD.

48, Toegyegongdan 1-gil, Chuncheon-Si, Gangwon-Do, Korea
Tel +82-31-441-2881 Fax +82-31-441-2883
E-mail vericom@vericom.co.kr overseas@vericomdental.com
Homepage www.vericom.co.kr



KTR Europe GmbH

Mergenthalerallee 77, 65760 Eschborn, Germany