

Coltène/Whaledent GmbH + Co.

 **KG** Raiffeisenstraße 30
89129 Langenau/Germany


www.coltene.com

STEP BY STEP CARD

HyFlex™EDM

 New file



 After autoclaving



 After use



 Twisted (discard)



Depending on the clinical situation, use of HyFlex EDM files reduces the number of files to 2-3 pieces particularly in straight and larger canals.



| | Optional Access Opening | Glidepath | | Shaping | Optional Finishing | | |
|----------------|-------------------------|-----------|----------------------|---------------------|--------------------|-------|-----------------------|
| CORONAL | | | | | | | |
| MIDDLE | | | | | | | |
| APICAL | | | | | | | |
| | 25/12 Orifice Opener | Hand File | 10/05 Glidepath File | 25/~ HyFlex OneFile | 40/04 | 50/03 | 60/02 Finishing Files |



| | Optional Access Opening | Glidepath | | Shaping | | Optional Finishing | | |
|----------------|-------------------------|-----------|----------------------|------------------|---------------------|--------------------|-------|-----------------------|
| CORONAL | | | | | | | | |
| MIDDLE | | | | | | | | |
| APICAL | | | | | | | | |
| | 25/12 Orifice Opener | Hand File | 10/05 Glidepath File | 20/05 HyFlex EDM | 25/~ HyFlex OneFile | 40/04 | 50/03 | 60/02 Finishing Files |

All HyFlex EDM NiTi Files can be used at 400 rpm and at a torque of up to 2.5 Ncm (25 mNm) excepted the Glidepath files which can be used with 300 rpm and at a torque of up to 1.8 Ncm (18 mNm).