Composite Tip Dispenser

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Description <p>The Composite Tip Dispenser is used to apply composite filling material from tips.</p>	
--	--

Intended use <ul style="list-style-type: none">Application of composite filling material from tips directly into the cavity being treated. Dispensing of composite filling material from tips onto a mixing block for application with an instrument	
--	--

Safety information <p>[WARNING]</p> <ul style="list-style-type: none">If not cleaned and sterilised properly, there is a risk of infection. Please read the instructions for reprocessing carefully and follow them.	
--	--

[CAUTION] <ul style="list-style-type: none">Only supplied to dentists and dental laboratories or upon their instructions. The dispenser is designed for composite tips from COLTENE. Users are responsible for checking whether other composite tips are compatible. If the dispenser is not handled properly, there is a risk of injury due to squashing. See chapter application.	
--	--

 <p>[NOTE]</p> <ul style="list-style-type: none">Follow the instructions for use for the composite filling material.	
---	--

Application <p>1. Prepare the dispenser (see chapter reprocessing). 2. Press the tip from below into the recess on the dispenser until it audibly clicks into place. 3. If necessary, turn the tip in the dispenser so that the end of the tip faces the desired direction. 4. Remove the sealing cap. 5. Δ Dispense the material by smoothly and consistently pulling the trigger. 6. Slowly release the trigger until it has reached its starting position again. Remove the tip from the recess and discard along with any residual material.</p>	
---	--

Reprocessing <p>Δ The dispenser should be reprocessed in accordance with EN ISO 17664:2004. The user is responsible for ensuring that the reprocessing is carried out correctly.</p>	
---	--

Preparation

• Application of composite filling material that has not cured with a clean soft cloth. If you cured composite filling material has adhered to the dispenser, place the dispenser in an alcohol-based cleaning fluid and then remove the material with a non-abrasive tool (e.g. a plastic spatula or a nylon brush). Do not use metal brushes or similarly harsh cleaning tools.

Manual cleaning and disinfection in an ultrasonic bath

For cleaning in an ultrasonic bath, place the contaminated dispenser in a basket inside the ultrasonic bath and immerse the dispenser in the cleaning fluid. Follow the dosing instructions for the cleaning solution. Start the ultrasonic cleaning process. Then remove the dispenser from the ultrasonic bath and rinse thoroughly under running water (preferably use deionised water). Dry the dispenser with compressed air.

Automated cleaning and disinfection in a washer-disinfector

If a washer-disinfector is available, preference should be given to auto-mated cleaning. Place the dispenser in the washer-disinfector in accordance with the recommendation of the equipment manufacturer. Select suitable cleaning fluids in accordance with the specifications of the product and equipment manufacturer and start the cleaning and disinfection process. Remove the dispenser from the washer-disinfector and, if required, dry with compressed air.

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